

Actilite®

Non-adherent viscose net dressing coated with 99% Manuka honey and 1% Manuka oil


Product description

Actilite® is a light viscose net dressing coated with antibacterial Manuka honey & Manuka oil. The dressing is designed to protect a wound, promote healing and allow the passage of exudate. The antibacterial effect of Actilite® has been enhanced by combining high grade antibacterial Manuka oil with Manuka honey.

The combination of Manuka honey and Manuka oil has been demonstrated in-vitro to be effective against a number of major wound infecting organisms including MRSA, VRE and Providentia stuartii.

Use

Remove from the outer packet, remove clear films and apply to the wound either side up. Apply a secondary dressing of choice, this can be an absorbent dressing or semi-permeable film dressing depending on the nature of the wound. We recommend Eclipse Adherent® or Advazorb Border®, Actilite® can also be used under Negative Pressure Wound Therapy (NPWT) or a primary layer to reduce bacterial burden.

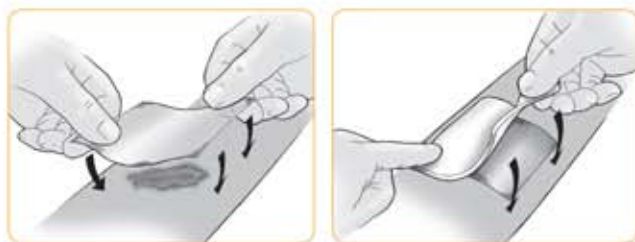

Contra-indications

Honey is not absorbed into the blood stream, however, we advise monitoring the levels of patients with diabetes. Do not use if allergic to bee venom. Discomfort can occasionally be experienced when honey is initially applied, it may be necessary to consider an appropriate level of analgesia. If discomfort continues, discontinue use and irrigate the wound with saline solution.

Storage

Store at room temperature. Due to the nature of honey, it can solidify at cold temperatures become more liquid at warm temperatures. If the product has hardened, warm between hands to soften before use. If the product has become too liquid, place in a colder place such as a fridge for a few minutes.

Actilite® ordering information

Size	Stock code	Pack size	NHS code	PIP code
5cm x 5cm	CR4281	10	EJE079	379-3759
10cm x 10cm	CR3849	10	EJE042	335-4917
10cm x 20cm	CR3852	10	EJE040	335-4925

TOP TIP

Actilite® is a light non adherent, non sticky dressing ideal for infection prevention.



Features

-  **Antibacterial**
-  **Osmotic effect**
-  **Enhanced healing**
-  **Odour reduction**
-  **Conformable**
-  **Non-adherent**

Indications

Suitable for all wound types where a primary layer is indicated and an antibacterial effect may be advantageous including:

-  **Cuts**
-  **Abrasions**
-  **Burns**
-  **Surgical wounds**
-  **Leg ulcers**
-  **Pressure ulcers**
-  **Diabetic ulcers**
-  **Infected wounds**

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilized using radiation		Read instructions for use

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

