HEINE QUALITY
MADE IN GERMANY

∴ HEINE BETA® Handles

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 0313 2013-09-20

HEINE BETA® Handles

DEUTSCH	3
ENGLISH	7
FRANÇAIS	11
ESPAÑOL	15
ITALIANO	19
SVENSKA	23
NEDERLANDS	27
SUOMI	31
PORTUGUÊS	35

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Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criterions: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Intended Use

BETA handles are designed exclusively for use with medical diagnostic instruments with bulb illumination by medical professionals.

Setting up the HEINE BETA® handles

Insert the batteries:

To insert the batteries (LR14), unscrew the end cap of the handle and insert

2 new batteries with the plus contact facing the handle head (see picture).

Application with automatic connector (AV):

Attaching the instrument:

Attach an instrument head by inserting it from above so that the connector lugs mate with the grooves in the handle. Press the head into the handle until it engages with an audible click.

Removing the instrument:

Remove the head by twisting the knurled ring with thumb and forefinger of the hand holding the handle. Pull off the head with the other hand.

Application with Twist-Lock-Connector (TL):

Attaching the instrument:

To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

Removing the instrument:

To remove the head rotate it anti-clockwise and pull it off.

ON/OFF switch and brightness control:

Press the button and rotate the black ring clockwise.

Using the BETA handles

BETA handles consist of three basic components:

- Handle module
- Rechargeable 3.5 V battery or batteries
- Bottom insert

The handle module (sleeve with head) is the same for all handles. The battery and bottom insert determine the handle type. By changing the bottom insert and battery the handle can be converted from one model to another.

- ⚠ The handles should not be used in rooms where an explosion hazard is present.
 - ⚠ Handles fitted with rechargeable batteries should not be used when being charged in the charger and should be switched off before recharging.
 - ⚠ The voltage shown on the identification plate of the mains transformer or bottom insert of the BETA R 3,5V handle must be identical to the local mains voltage.
-  The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries are used.

System overview

1. HEINE BETA® battery handle

Use alkaline "C"-size batteries. With the BETA extension piece a third battery can be added to extend the operation (4.5V).

2. HEINE BETA® NT rechargeable handle:

You can fit either NiMH or Li-ion rechargeable batteries.

3. HEINE BETA® cordle handle:

The cord handle contains no battery. It may only be connected to 6 V maximum supply.

4. HEINE BETA® R rechargeable handle:

The bottom insert incorporates a cap to protect the plug pins. This should be screwed on after charging. You can fit either NiMH or Li-ion rechargeable batteries.

Technical specification:

Mains supply:	230 V/50–60 Hz
See bottom insert:	120V/60 Hz
Power consumption:	max. 2 VA
Safety class:	double insulated
Recharging current:	max. 75 mA

5. HEINE BETA® TR rechargeable handle:

Use only the charging transformer supplied with the handle. During charging the green LED lights up to indicate that a charging current is flowing. You can fit either NiMH or Li-ion rechargeable batteries.

Technical specification:

Mains supply:	100–240 V/50–60 Hz
Power consumption:	max. 9 VA
Safety class:	double insulated
Recharging current:	max. 75 mA

Technical specification

<i>Environmental conditions for operation:</i>	
Permissible Ambient temperature:	+10°C to +40°C
Rel. humidity:	10% to 75% rel. humidity
Air Pressure:	700hPa to 1060 hPa
<i>Environmental conditions for transport and storage:</i>	
Temperature:	-20°C to +50°C
Rel. humidity:	10% to 95°
Air Pressure:	500 hPa to 1060 hPa
Power range:	DC 2.5V – 4.5V

Instructions for rechargeable battery handling

Spontaneous discharge can occur when the battery is not recharged over a period of weeks. In this case a 14-hour charge is enough. Replace the battery when a 14-hour charge is no longer sufficient to guarantee enough working capacity.

Cleaning and disinfection

The components can be wiped with a cloth moistened with water, cleaning solution or disinfectant.

Ensure that the handle is dry after cleaning.

Disconnect from the mains supply before cleaning BETA NT handle, BETA R handle and BETA TR handle.

Maintenance

Changing the battery:

- Unscrew the bottom insert from the handle module.
- Remove the batteries or rechargeable battery.
- Insert the new battery.
- Screw the bottom insert into the handle module.

Service

The device does not require regular service.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

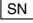
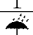
Recycle batteries separately.

General notes and warnings

-  The device is not allowed to be entered into strong magnetic fields like MRI scanners.
-  The performance and safety of the device can only be guaranteed when fitted with original HEINE accessories and HEINE spare parts. Otherwise the warranty is terminated.
-  Don't modify the device! If the device is modified despite that, take care that use of the device is safe by performing investigations and tests.
-  All repairs other than a change of batteries should be carried out by qualified professionals.
-  Check the correct operation of the BETA handles before usage. Do not use the equipment if it shows visible signs of damage.
-  Do not drop the handle from a great height. It can damage the product and/or operator.
-  Never open the bottom insert or the transformer.
-  Moisture must not be allowed to enter the handle and/or the bottom insert.

Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.
	Catalogue- or reference number
	Serial number
	Manufacturer
	Date of manufacture
	Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Pressure limitation for storage and transport
	Please read and follow the instructions for use and keep them for future reference (background color blue; foreground color white)
	Fragile, handle with care!
	Keep dry
	Interference may occur in the vicinity of equipment marked with the following symbol
	"Grüner Punkt" (country-specific)

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