HEINE QUALITY

MADE IN GERMANY

HEINE BETA® 200

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 2 02
E-Mail: info@heine.com · www.heine.com
med 5996 **08.2013**

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General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposable, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle. The guarantee covers irrepairable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

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Instructions

Ophthalmoscope HEINE BETA® 200

 Before using the Ophthalmoscope HEINE BETA® 200 read these instructions carefully and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: This instrument is designed for examination of the eye.

Safety in use: Use only handles or devices which are specifically designed for medical use as a power supply. The supply voltage must be the same as bulb voltage.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Key to Illustrations

- Spectacle rest
- Dustproof viewing aperture
- Lens wheel
 - +dpt: 1-10 | 15 | 20 | 40
 - dpt: 1-10 | 15 | 20 | 25 | 35
- Diopter readout (plus = green, minus = red)
- Thumbrest
- HEINE XHL® Xenon halogen bulb
- Locating pin
- Dustproof window
- Aperture wheel
- Filter selector (an interference red-free filter can be combined with any aperture)

Using the instrument

To ensure maximum user comfort, hold the instrument so that the thumb is located on the thumbrest (5). This makes it possible to operate the lens wheel (3), aperture wheel (9), and filter selector (10).

Changing the bulb

Please note: The performance of this instruments can only be guaranteed if genuine HEINE bulbs are used.

Simply detach the ophthalmoscope from the handle and pull the bulb out. Clean the glass of the new bulb with a soft cloth. Insert the bulb fully into the body of the instrument. The pin must locate in the groove.

Cleaning

The instrument housing can be cleaned with a cloth moistened with alcohol. Clean the glass of (2) and (8) and the diopter readout (4) with a cotton-wool bud moistened with alcohol. The tip should be rotated without excessive pressure in a spiral from the centre to the edge of the window.

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