## HEINE Otoscopes

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)8152/38-0  
Fax +49(0)8152/38-202  
E-Mail: info@heine.com · www.heine.com

med 0514 2014-10-08

V-200.00.151

### DEUTSCH

#### HEINE Otoskope

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### HEINE Otoscopes

These instructions apply to the following products of the HEINE Otoscope series: BETA200, BETA400, K180, mini3000 F.O., mini3000 F.O. LED, mini3000, BETA100, K100.

Please read and follow these instructions for use and keep them for future reference.

#### Intended Use

HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have an optical magnifying equipment and a battery-rechargeable battery-powered illumination equipment. The instruments may only be used by qualified medical personnel in medical rooms. Patients can be investigated as often as necessary without further restrictions. The devices are intended for short term treatment and can also be used for instrumentation with nonactive medical-devices.

#### For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

#### Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

#### Maintenance and Service

The instruments do not require maintenance or service.

#### General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not shine light of the instrument directly into the eyes.

#### General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

If you don't use the device for a longer period of time, please remove the batteries in advance.

#### Disposal

The product must be recycled as separated electrical and electronic devices.

Please observe the relevant state-specific disposal regulations.

#### Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

This device is intended for use by medical professionals in the electromagnetic environment specified below. The user of this device should assure that it is used in such an environment

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices.

If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains the tables „Guidance and Manufacturer's declaration – Electromagnetic immunity“, technical data and an explanation of the used symbols.

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