
## HEINE BETA® Batteriegriff HEINE BETA® Battery handle

CE 96 73/23/EWG  
89/336/EWG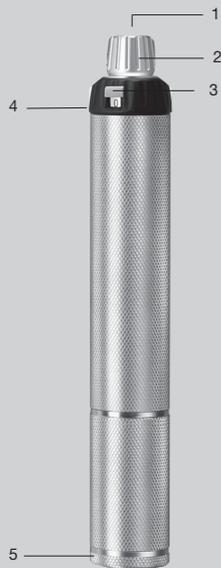

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52/38-0  
Fax +49 (0) 81 52/38-202  
E-Mail: info@heine.com · www.heine.com  
tec 4496 1/11.10

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CE 96 73/23/EEC  
89/336/EEC

## Instructions

Read these instructions carefully before using the Battery handle and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

### Safety information

**Range of applications:** The battery handle are designed exclusively for charging HEINE instruments with 3,5 V bulb illumination.

**Safety in use:** The battery handle can be powered by 3 batteries or by LR 14 – size rechargeable batteries.

### Using the handle

**Connecting the instrument:** The instrument is inserted from above so that the lugs of the automatic connector mate with the grooves in the head of the handle (1). Press the instrument into the handle until it engages with an audible "click".

**Disconnecting the instrument:** If the knurled ring (2) is twisted to the right with the thumb and forefinger of the hand which is holding the handle, the instrument can be withdrawn upwards with the other hand.

**Brightness control:** Press the button (3) and rotate the black ring (4) clockwise. Position "1" represents maximum illumination. Between "0" and "1" the desired value can be selected according to individual requirement.

**Changing battery:** The batteries can be removed after unscrewing the cap (5).

**Applicable standards:** EN 60 601-1  
EN 55011

### Warranty

We guarantee this device (except bulbs) for two years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from

date