NEBULFLAEM® F2000

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Aerosol therapy is one of the methods of drug delivery for the treatment of chronic and acute pulmonary disease. This form of treatment is increasingly favoured by both family doctors and specialists since it achieves maximum drug efficacy directly in the respiratory tract thus minimising possible side-effects. The most common method of generating an aerosol is by directing compressed air from a compressor into a nebuliser which then converts liquid into droplets based on the venturi principle. Nebulised drug is inhaled by using one of the following accessories: mask, nasal prong or mouthpiece.

Nebulflaem F2000 is designed for heavy use to meet the strict requirements for hospital and clinical use. The **Nebulflaem F2000** has a powerful **reciprocating piston compressor** which requires no lubrication (even when used continuously).

The **Nebulflaem F2000** system incorporates the **RAPIDFLAEM 4** jet nebuliser and a full set of hygienically plastic-wrapped accessories including mouthpiece, adult mask, two soft child masks in colors blue and pink and nasal prongs.

Flaem's patented anti vibration system offers a silent and vibration free operation which makes it comfortable for use.

The **Nebulflaem F2000** system provides continuous high performance comparable to larger and more expensive units even after frequent and prolonged use.

The case is moulded from thermoplastic resins, has robust features, a handy carry handle and a large accessory compartment which is hygienically separated from the cord storage compartment. **CLINICALLY TESTED.**

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The definition for **aerosol** is "a suspension of solid and liquid particles in a gaseous medium". The deposition and settling of these particles on the mucosae and their penetration into the respiratory tract is dependant on their diameter which according to current scientific studies states that the ideal range is between 0.5 to 8µm.

The particles size delivered by the nebuliser is measured by means of sophisticated computer controlled laser device which has high-precision measuring property enabling it to calculate a value defined as MMAD (Mass Median Aerodynamic Diameter). A nebuliser is considered efficient if its MMAD is between 0.8 and 5µm.

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** Certified with TSI Aerosizer 3220 computerised laser system in compliance with Flaem internal procedure.

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FLAEM NUOVA S.p.A
FLAEM NUOVA S.p.A. è un'azienda certificata UNI CEI EN 46002.
The Company **FLAEM NUOVA S.p.A.** is UNI CEI EN 46002 certified.