HEINE QUALITY
MADE IN GERMANY

HEINE mini 3000®
Diagnostic Sets
D-851, D-852, D-856, D-873

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 202
E-Mail: info@heine.com · www.heine.com
med 4706 17.12

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DEUTSCH

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Instructions for use

ENGLISH

Read these instructions carefully before using the HEINE mini 3000® and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information according to the international standard IEC 60601-1 “Electric Safety of Medical devices”

Do not open the instrument / battery handle in the vicinity of the patient.

1. HEINE mini 3000® battery handle

Range of applications: The HEINE mini 3000® battery handle should only be used for operation to power HEINE mini 3000® instruments.

Batteries:

- 2 alkali cells (AA/LR 6)
 - Important:** Do not use alkali manganese batteries with lithium as bulb life will be reduced.
 - The HEINE mini 3000® battery handle can be charged in the HEINE mini NT® with the HEINE mini 2Z rechargeable battery 2,5 V.
- Charging time:**
- The mini NT indicates when the rechargeable battery is fully-charged. The illuminated ring is lit continuously.
 - mini 2Z rechargeable battery (2,5 V, NiMH = ca. 4 h (730 mAh)

Instrument connector: Internal thread in handle head (1).

Switching on: Push switch (2) down.

Switching off: Push switch up. If the handle is put back firmly into a coat pocket the handle is automatically switched off.

Changing the batteries: Unscrew the end-cap (3) and shake out the old batteries/rechargeable battery. Insert new batteries/ rechargeable battery with the plus contact facing the handle head.

Cleaning: The handle can be cleaned with a damp cloth (e.g. alkaline or pH-neutral detergent). For cleaning disinfection of the outer surface we recommend the so called wipe disinfection by using detergents, which are released for Medical Devices made out of plastic. Disinfection by spraying or immersion as well as sterilisation is not allowed.

Important note:

We can only guarantee the performance and safety of HEINE mini 3000® instruments when they are fitted with original HEINE bulbs.

2. HEINE mini 3000® Ophthalmoscope

Range of applications: The HEINE mini 3000® ophthalmoscope is designed for examination of the eye. During examination, hold the instrument with your index finger on the lens wheel (4). You can also operate the aperture wheel (6) in this way. Window (5) shows the selected lens value. Minus lenses are shown red, plus lenses black.

Changing the bulb: Unscrew the instrument from the handle and pull the bulb (7) downwards. Wipe the new bulb glass clean and insert it so that the locating pin (8) fits the slit.

Cleaning: The housing can be wiped clean with a cloth moistened with alcohol. Glass surfaces can be cleaned with a cotton wool bud in the same way.

3. mini 3000 Otoscope/mini 3000 F.O. Otoscope

Range of applications: The HEINE mini 3000® otoscope should only be used to examine the external ear, the nose and for general non-invasive examination.
Safety in use: The otoscope should only be used to examine the ear or nose when a tip is fitted. The contact time (skin) has to be limited to max. 1 minute.

Use

Attaching a tip: The otoscope may only be inserted when a disposable (grey) or reusable (black) tip is fitted. Push the tip onto the instrument so that the internal projection engages in the slit (9). Twist to the right to lock.

Magnifying lens: The lens (10) magnifies ca. 3x and can be swivelled aside for instrumentation.

Insufflation port: mini 3000 F.O. Otoskop: An insufflation bulb can be fitted to the port.

General illumination: The otoscope can be used with or without a tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

Changing the bulb:

HEINE mini 3000® otoscope: Swivel the lens (10) to one side and simply pull the bulb backwards out of the head.

With the **HEINE mini 3000® F.O. otoscope**, first unscrew the head from the handle and pull the bulb (10a) downwards. Push in the new bulb.

Cleaning and Sterilization:

Otoscope head: Clean the outside with a soft cloth, inside with a cotton wool bud, if necessary soaked in alcohol. Gas sterilization is permitted. Do not soak.

Reusable SANALON S tips: These can be cleaned, disinfected and sterilized or boiled by any conventional method. Autoclaving is permitted.

AllSpec-Tips: Are for single use only. Do not attempt to clean them as the tip may be damaged and could injure the patient.

4. HEINE mini 3000® laryngeal mirror

Safety informations

Range of applications: The straight laryngeal mirror is used to illuminate the mouth and throat during medical examination.

Safety in use: The curved light stem should never be inserted into the mouth without an attachment fitted to the end as the bulb can become hot. Switch off the instrument after use to avoid it becoming too hot.

Using the instrument: To avoid fogging of the mirror during use, the warmth from the bulb is usually sufficient. We recommend that the instrument be switched on a few seconds before the examination is made.

Changing the bulb: Unscrew first the mirror and then the bulb from the light stem.

Cleaning and sterilizing:

Do not place the light stem in liquid.

- Cleaning:** Unscrew and remove the mirror from the light stem and clean both parts with a damp cloth – we recommend a soapy solution or alcohol.
- Disinfection:** After cleaning, wipe all parts with a conventional disinfectant. Instrument heads can be soaked in disinfectant solutions, but should always be dried before use.
- Sterilization:** Gas sterilization with ethylene oxide is recommended for all components. Autoclaving **should not be used for the mirror and light stem.**

Applicable standards:
ISO 15004 -1 (Ophthalmic instruments - Fundamental requirements and test methods)

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