




# ABSORBENT DRESSING WITH STRIKE-THROUGH EXUDATE BARRIER


Mesorb is a soft highly absorbent dressing that can retain large amounts of wound exudate. The fluid-repellent strike-through barrier helps prevent exudate soiling clothing and bed linen.

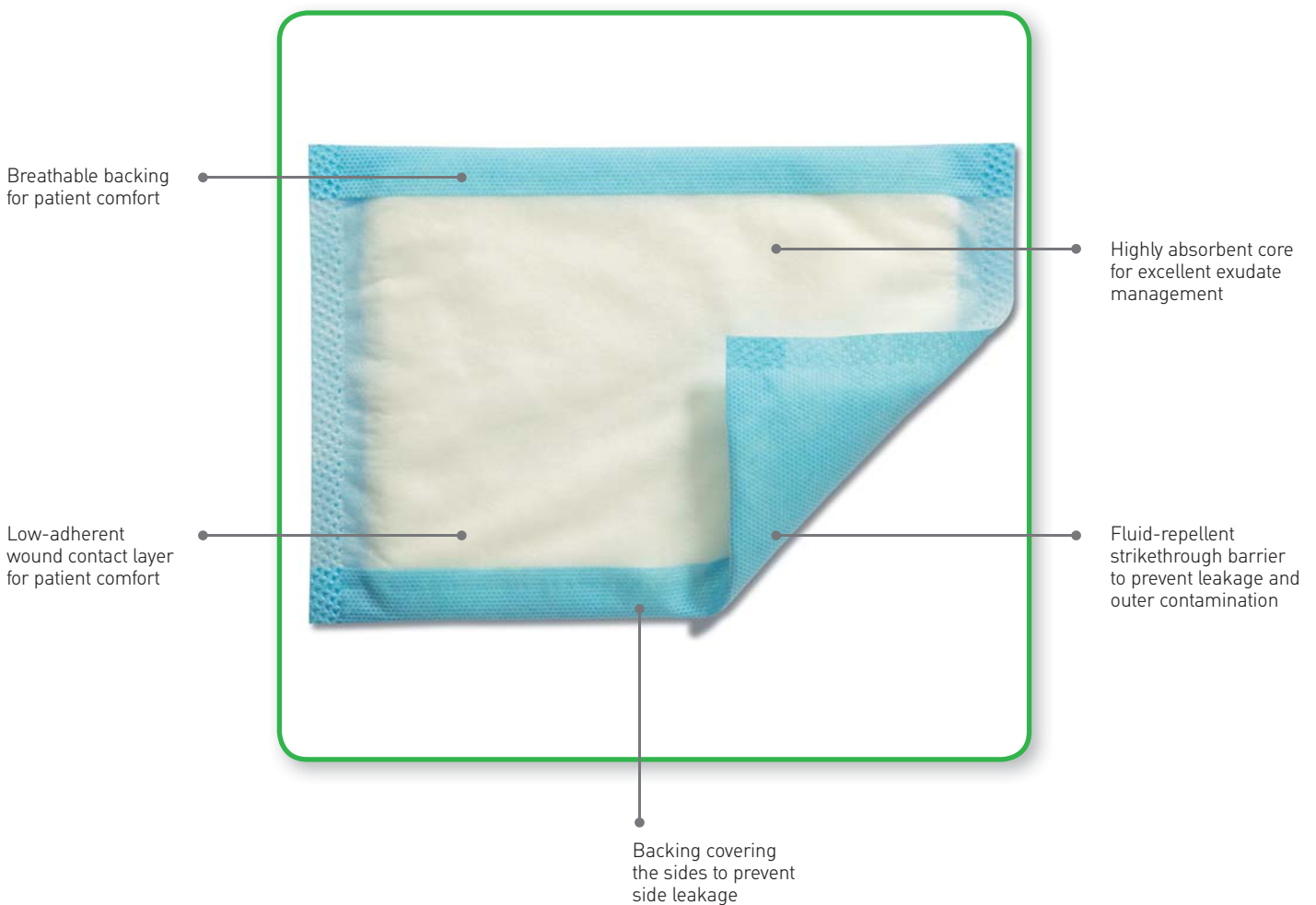

## HOW MESORB WORKS

Wound exudate is absorbed and transferred into Mesorb by the wound contact layer and the diffusion layer. The highly absorbent core absorbs exudate, while simultaneously providing a good protective and ventilating wound cushion. The fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. As the backing covers the sides of the dressing, it also prevents side leakage.


## HOW TO USE MESORB

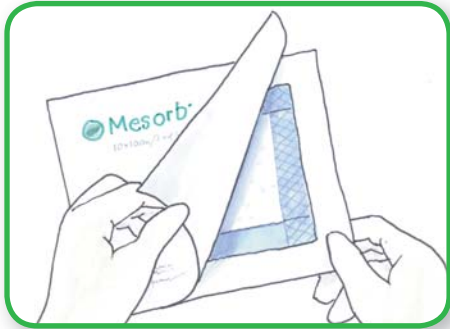

1. Open the package and remove the dressing.



2. Apply Mesorb on the wound area or on the chosen primary dressing with at least 1 cm overlap on the surrounding skin.

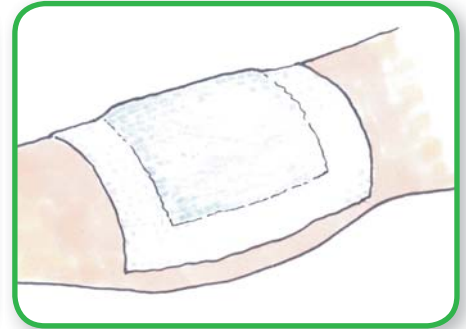

3. Fix in place with a suitable fixation device (e.g. Mefix®, Mepore® Film, or Tubifast™)

## FREQUENCY OF CHANGE

Mesorb may be left in place for several days, depending on the wound and skin condition, or as indicated by a healthcare professional.

## BENEFITS OF MESORB

- Highly absorbent
- Good protective cushioning
- Breathable
- Cost effective
- The unique strike-through exudate barrier reduces frequency of dressing changes and prevents soiling of clothing and bed linen



## PRECAUTIONS

Mesorb should not be used on infected wounds without consulting a health care professional.

### MESORB ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
677080	10 x 10	10	225-3527	EJE044
677280	10 x 15	10	225-3535	EJE056
677480	10 x 20	10	225-3543	EJE058
677680	15 x 20	10	225-3550	EJE059
677780	20 x 25	10	225-3568	EJE060
677880	20 x 30	10	225-3576	EJE061



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, [www.molnlycke.co.uk](http://www.molnlycke.co.uk)

The Mölnlycke Health Care name and logo, Mesorb® are registered trademarks of Mölnlycke Health Care AB  
© 2009 Mölnlycke Health Care. All rights reserved.

UK117550611