




Please read this leaflet before you begin the test


The **Yes! Cassette Pregnancy Test** detects hCG, the hormone produced during pregnancy. The amount of this pregnancy hormone increases as your pregnancy progresses. This test is designed to detect 25mlU/ml of the hCG and should be able to tell you if you are pregnant the day your period is due if you have a 28-day cycle.

SIMPLE TO USE: It is best if you test your urine first thing in the morning. This is when the levels of the pregnancy hormone show higher.

DIRECTIONS

- 1. Collect urine in a clean and dry container.
- 2. Remove the test from the pouch and use it within one hour.
- 3. Place the test cassette on a clean and level surface.
- 4. Hold the dropper vertically and transfer 3 full drops of urine to the specimen well of the test cassette, and then start the timer.
- 5. Avoid trapping air bubbles in the specimen well (see illustration below).
- 6. Wait for the colour line(s) to appear.
- 7. The result should be read at 3 minutes, if no coloured line, wait 1 minute longer.
- 8. Do not read the result after 10 minutes.

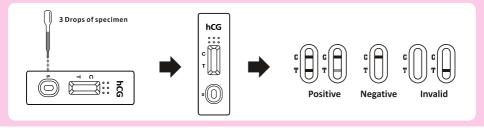

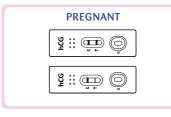



How to interpret the result and next steps

PREGNANT - A pregnant result means your urine contains hCG, a hormone your body makes when you are pregnant. If your test says you are pregnant you should see your doctor who can advise you on what steps you should take next.

NOT PREGNANT - You may not be pregnant, or the level of pregnancy hormone may not yet be high enough to be detected, or you may have miscalculated the day your period was due. If you tested early, test again when your period is due. If your period is overdue, test again after 3 days. If you still get a **not pregnant** result but your period is still late then see your doctor because there could be health-related reasons as to why you haven't had your period.

QUESTIONS & ANSWERS



How does the test work?

The Yes! Cassette Pregnancy Test detects hCG, the hormone produced during pregnancy. The amount of this pregnancy hormone increases as your pregnancy progresses. This test is designed to detect 25mlU/ml of the hCG and should be able to tell you if you are pregnant the day your period is due if you have a 28-day cycle. The level of hormone is greater first thing in the morning so it is best to test your first urine of the day.

How accurate is the test?

This test is over 99% accurate if used 7 days after your next period is due. Hormone levels in women do vary and some women may not have detectable levels of the hormone on the first day they test for it.

What do I do if the result says I am pregnant?

A positive pregnancy test result shows that levels of hCG have been detected. You should see your doctor to confirm, as this test is so sensitive some women may test positive even if they are not pregnant (this is usually due to a hormone called pituitary hCG or a medical condition such an ovarian cyst or ectopic pregnancy). It is also important for you to receive medical advice for proper care and counselling.

What do I do if the test says I am not pregnant?

This would mean you may not be pregnant or it is too soon because your levels of hCG are too low. You may also have miscalculated your menstrual cycle, or when you thought you may have conceived. If you still haven't had your period, or still think you are pregnant, then retest with a second test after three days. If you retest, and the result is still negative, then you should contact your doctor as there may be a health related reason for why your period is not happening.

What happens if the control line doesn't come up?

The result should appear within three minutes. If not wait another minute. If the lines have not appeared after 10 minutes or the control line is not present, then the test is invalid. You should

retest with a new test, carefully following the instructions.

What if the test line is very faint?

Even if the test line is faint it means a positive pregnancy result.

Will the result change if I leave it for some time?
The test needs to be read after 3 minutes but within 10 minutes

Do drugs or medication affect the test?

This test should not be affected by hormone therapies containing clomiphene, by alcohol, painkillers, antibiotics or the contraceptive pill. It will only be affected by drugs containing the pregnancy hormone hCG.

I have tried getting pregnant but have not yet succeeded. Is there anything I can do?

You are most likely to get pregnant during the 2-3 days around when you are ovulating. To be able to tell when this is happening use the Yes! Baby Now Ovulation Test or the Yes! Fertility Test. This kit contains all you need to have the best chance of conceiving- ovulation tests, pregnancy tests, a sperm concentration test and an ovulation chart and special ovulation thermometer.

Who can I contact if I have more questions?

For further information or advice on using the Yes! Rapid Response Pregnancy Test, either contact the pharmacy you purchased the tests from *OR* contact Smith Biomed (NZ) Ltd on 0508 BIOMED (246 633). Address all emails to info@smithbiomed.com.

Further product information sheets are available at www.smithbiomed.com